

SIEMENS

MAGNETOM

MR

Function Description

System

System

© Siemens 2008

The reproduction, transmission or use of this document or its contents is not permitted without express written authority. Offenders will be liable for damages. All rights, including rights created by patent grant or registration of a utility model or design, are reserved.

Print No.: M6-040.850.10.01.02
Replaces: n.a.

English
Doc. Gen. Date: 03.08

Document revision level

The document corresponds to the version/revision level effective at the time of system delivery. Revisions to hardcopy documentation are not automatically distributed.

Please contact your local Siemens office to order current revision levels.

Disclaimer

The installation and service of equipment described herein is to be performed by qualified personnel who are employed by Siemens or one of its affiliates or who are otherwise authorized by Siemens or one of its affiliates to provide such services.

Assemblers and other persons who are not employed by or otherwise directly affiliated with or authorized by Siemens or one of its affiliates are directed to contact one of the local offices of Siemens or one of its affiliates before attempting installation or service procedures.

1	Introduction	5
	Documentation structure	5
	Hardcopy documentation	5
	CB-DOC	5
2	System Introduction	7
	Overview	7
	Magnet System	9
	Magnet	9
	Magnet Supervision - MSUP	9
	Magnet Covers	9
	Shim Power Supply	9
	Passive Shim	9
	Magnet Power Supply - MPS	9
	Array Shim Device	10
	Gradient System	11
	Gradient Amplifier - GPA	11
	Gradient Coil - GC	11
	RF System	12
	RF Small Signal Unit - RFSU	12
	RF Power Amplifier - RFPA	12
	RF Application System - RFAS	12
	RF Infrastructure - RFIS	12
	RF Filter	12
	Body Coil - BC	13
	Local Coils - LC	13
	Spectroscopy	13
	Patient Handling	14
	Patient Table - PTAB	14
	Patient Trolley	14
	Intercom	14
	Patient Monitoring	14
	System Control	15
	Host Computer - HOST	15
	Image Reconstruction System - MRIR	15
	Measurement Control - AMC	15
	Physiological Signal Acquisition - PMU	15
	In-Room MRC	15
	System Supply (mains, cooling water)	16
	Line Power Distribution - PDS	16
	Air Conditioning System - ACS	16
	Cooling System	16
	Separator Cabinet - SEP	16
	Chiller - COCS	16
	Interface and Filter Panel - IFP	16
	System Cabling	16
	Software	17

3	List of Abbreviations	18
4	Changes to previous version	25

Documentation structure

This prefix is intended to familiarize you with the structure of the MAGNETOM Verio documentation. This prefix explains the print format as well as the on-line documentation and how to use it.

Hardcopy documentation

The hardcopy service documentation will be issued in DIN A5 portrait format and comprise a Functional Description (**FUN**), the Troubleshooting Guide (**TSG**) and the Replacement of Parts (**REP**).

Changes to the previous version

The last part of each chapter shows the changes to the previously released version. You will find the topic, section or graphic which was changed together with the corresponding page number.

CB-DOC

As a consistent feature, the documentation has been put onto electronic media - the Computer Based Documentation or CB-DOC - which is delivered on DVD media.

The electronic documents are identical to the hardcopy versions and available in both HTML and PDF formats. To read the HTML files, you will need a browser installed on your service PC (recommended: **MS Internet Explorer**). Viewing PDF files requires Adobe Acrobat Reader.

Navigation in and between the various electronic documents is made possible with hyperlinks which can be recognized by their BLUE color.

Documents on the CD

Files included in html or pdf format:

- Functional Description
- Troubleshooting Guide
- Replacement of Parts
- Planning Guide
- Installation
- Start-up
- Diagrams
- Maintenance Instructions
- Safety Information

Service PC

As a service engineer you will be supplied with a portable service PC to access all relevant information.

Documentation access at the MR system

The electronic documentation can also be used at the MR system after inserting your DVD into the CD/DVD-ROM drive of the Host.

以上内容仅为本文档的试下载部分，为可阅读页数的一半内容。如要下载或阅读全文，请访问：<https://d.book118.com/798012040052006114>